

Y-700 RF Wireless Call System MEEYI

Y-700 is the receiver part of the wireless call system, as a wireless nurse call lamp for indicating call information to caregivers remind them provide in-time service to patients.

Y-700 wireless call system corridor lamp can be widely used in hospitals, nursing homes, hotels, office buildings, schools, parking lots, and so on.

What's Y-700 ?

Y-700 is the receiver part of the wireless call system, as a wireless nurse call lamp for indicating call information to caregivers remind them provide in-time service to patients.

MAIN FUNCTIONS

- Can work with 90 transmitters in max
- One color can work with 30 transmitters in max
- Can set each button with different colors

HOW TO INSTALL?

Y-700 wireless call system corridor lamp is usually placed by the door or above the door to remind caregivers not to miss any call information.



TECH SPEC

Color of Light	Total 3 colors (Red, Yellow, Green)
Ringtones	The sound level can be adjusted (large, medium, small)
Power Supply	DC 12V 1A
Alarm light Visible Distance	50 meters
Dimension	104 (W) x86 (H) x38 (D) mm
Operating Frequency	433.92MHz

I. System Feature:

Using three colors(Red,Yellow,Green) of LED light,Y-700 is used to indicate different service type:






Press “call” button on the transmitter (Model: Y-A1 or Y-A3 or Y-SC), Y-700 flashes in red color.

Press “bill” button on the transmitter (Model: Y-A3),Y-700 flashes in green color.

Yellow light is empty.

Y-700 can work with 90 transmitters(Model: Y-A1 or Y-A3 or Y-SC) in max.

II. System Consist:

	Product	Model	Dimension(mm)
	Wireless corridor light	Y-700	104x86x38
	1-key transmitter (call button)	Y-A1-WR	Φ60x60x17
	1-key transmitter (cancel button)	Y-A1-BC	Φ60x60x17
	3-key transmitter	Y-A3-WR	Φ60x60x17
	3-key transmitter	Y-A3-WG	Φ60x60x17

III. System Instruction:

1. Function key(Open the casing,on the board):

Under setting status:

- (1)Red Key(R):save/delete
- (2)Yellow Key(Y): save/delete
- (3)Green Key(G): save/delete

Under working status:

- (1)Red Key(R):cancel
- (2)Yellow Key(Y):cancel

(3)Green Key(G):cancel

2.Function setting:

- (1)power on: a. “flashing” means it is under working status
b. “Flashing non-stop” means it is in fault.

(2)Save transmitter: power on, press the Red Key(R),then press the “call button” on the transmitter(Model: Y-A1 or Y-A3 or Y-SC) for 3 seconds, will hear “Di Di Di” with flashing,which means it's successful to save the transmitter. Press next “call” button or “cancel” button to save the next transmitter(90pcs in max).

Release “R” means exit the setting status.

“Di Di Di Di Di Di” non-stop means no space for more transmitters.

“Di Di” means the transmitter has been saved already “Di Di Di” means the transmitter has been saved successfully.

Same with green & yellow light.

Note: 1.Any key of the above is fine for saving transmitters.

2.Only one “Cancel” button can cancel all the transmitters.

(3)Delete transmitter: press the Red Key(R)for 3 seconds.

Note: Saving transmitters should be done completely. After saving,when press the Red Key(R)again,it will delete all the saved transmitters.

(4)The volume of sound is adjustable: high/ mute

IV. Technical Parameter:

Working voltage : DC12V 500mA

Working current: <65mA

Standby Current: <10mA

Sensitivity: <5uV (-106dB)

Volume Output: 85dB

Working environment: -40~80℃ Humidity:R <80 %

Dimension:104mm(L)X86mm(W)X38mm(H)